



BACTINYL® DISINFECTANT WIPES

Single use disinfectant wipes





BACTINYL® DISINFECTANT WIPES



Cleaning and disinfection of non-invasive medical devices, instrumentation, materials (including sensitive surfaces) and medical equipment.



Unique gentle formula



Contains less than 5% alcohol



Designed for the disinfection of medical devices



2 versions : Perfumed with the scent of pine or odourless



Long lasting impregnation



Shelf life : 4 years



Tested and approved for hospital use



Non invasive
Medical devices
IIA Class





BACTINYL® DISINFECTANT WIPES

Activity	Contact time
Bactericidal	
NF EN 1040	5 min
NF EN 1276 – Dirty conditions (Pseudomonas aeruginosa, Escherichia coli, Staphylococcus aureus, Enterococcus hirae, Ralstonia pickettii)	5 min
NF T 72-171 – Dirty conditions	1 min
NF T 72-190	15 min
NF T 72-190 (Mycobacterium smegmatis)	5 min
Yeasticidal	
NF EN 1650 – Dirty conditions	15 min
Fungicidal	
NF EN 1275	60 min
Virucidal	
NF T 72-180 (Rotavirus, Poliovirus, herpes virus, HIV-1)	15 min
NF EN 14476 (H1N1)	1 min

INSTRUCTION FOR USE

Wipe the surface for 2 seconds with BACTINYL® DISINFECTANT WIPES.

Allow to dry. Do not rinse. Do not use on surfaces which may come into contact with foodstuffs.

Replace lid after use.

Does not replace disinfection by soaking.

PACKAGING

Tub of 120 wipes

Perfumed

Ref. 115-2080 - French - English - Spanish - German - Dutch - Portuguese

Ref. 115-2082 - French - Danish - Norwegian - Finnish - Swedish - Italian

Odourless

Ref. 115-2115 - French - English - Spanish - German - Dutch - Portuguese

Ref. 115-2122 - French - Danish - Norwegian - Finnish - Swedish - Italian

PRECAUTIONS

For more information, please refer to the Material Safety Data Sheet. Product for professional use only.



LABORATOIRE GARCIN-BACTINYL



Parc Industriel de la Plaine de l'Ain
225 allée des Cèdres
01150 St Vulbas – France
www.orapi.com

